BEYOND TOUCH  
A NEW FORM OF “SEEING”

# Cos'è AuxDeco

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## What AuxDeco is

AuxDeco will help the blind walk. Can distinguish contours of objects and landscape features as if visible without to touch the hand or cane. With the aid of an appropriate training program, will be able to walk safely on the road without Braille Block. It helps to improve their quality of life.

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## Configuration

Headband, processor and battery are connected with cables. Converted into electrical signals in a compact camera images taken with electrode plates directly stimulate the forehead is a device in real time, we feel that the situation in front. HEADBAND: installed a small camera and a series of small electrodes. PROCESSOR: small computer (14,5x7,8 cm). BATTERY: lightweight, 250 gr.

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## How the device works

### IMAGE CAPTURE

A small camera is mounted on the device. As the camera position is fixed, the user must turn his head in the direction he wants to see.

### DIGITAL PROCESSING

The computer converts the images to black and white, then extracts the image outlines. These outlines are converted into electrical impulses.

### TRANSFER TO TACTILE STIMULI

The converted impulses are sent out via the grid of 512 electrodes set on the user's forehead.

### MENTAL PROCESSING AND PHYSICAL INTERACTIOS

Using the impulses to sense position, movements and objects, the user is able to have a "view" of what is in front of him.

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## The Strength of AuxDeco

**IMPROVED DISTANCE.** Adjusting the camera through head movements, the user is potentially able to recognize directions and landmarks from a much greater distance.

**MOVEMENT RECOGNITION.** The system conveys moving object information to the user. A feature useful in high-traffic and noisy areas.

**2D PATTERN RECOGNITION.** The AuxDeco can accurately transmits intangible 2D patterns such as crosswalk stripes and other road markings. The user deals for navigating intersections. Also, AuxDeco provides a convenient safety in potentially dangerous or unsanitary surroundings, reducing the need for direct and physical contact.

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## Training

In order to use the forehead for tactile stimuli, the AuxDeco system needs no surgery, but does need specific training. After mastering the basic techniques of the system with the training, the user should be able to put the AuxDeco to practical, daily life use.

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AuxDeco European Launch October 2012, Isernia, Italy

## AuxDeco in the world

AuxDeco Announcement USA

NorTech Innovation Awards USA

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AuxDeco Sweden

World Innovation Summit SPAIN

## **EyePlusPlus Europe Ltd**

**EyePlusPlus Europe Ltd Zona Industriale - 86090 Pettoranello di Molise (IS) - ITALY  
Tel + 39 0865 460305 Fax +39 0865 410150 [eyeplusplus@advancedmedical.it](mailto:eyeplusplus@advancedmedical.it)**

**Peatling Lodge, Bruntingthorpe Road, Peatling Parva, Lutterworth - LE17 5RB U.K.  
Tel +44 (0) 870 751 889 Fax +44 (0) 870 751 8892**

**[www.advancedmedical.it](http://www.advancedmedical.it)**